

Participant Instructions for Viewing PreciPoint Scanned Images for The Haematological Malignancies Bone Marrow Aspirate Assessment Programme

PLEASE NOTE – The images for the HMBMAA programme are now being hosted by PreciPoint and will, therefore, appear differently to previous exercises.

1. To view the image, copy and paste the image link into your web browser in a new tab and the image will appear as below



2. There are a number of ways to zoom in on the image.

- a. The times magnification at the top left of the screen



- b. The – and +





- c. The FIT icon will return the magnification to fully zoomed out



- d. The 100% icon will immediately magnify the image to x100




- e. The scroll wheel on the mouse will zoom in or out of the image depending on which direction is scrolled


4. The  icon will show the image as full screen as shown below. Clicking on the  icon will exit full screen mode and return the page to the initial view




5. The cog icon (View settings) will allow addition and removal of the minimap, the scale bar and the magnifications.

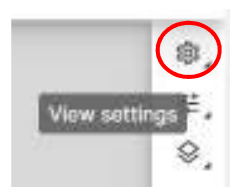

– these can be toggled on and off

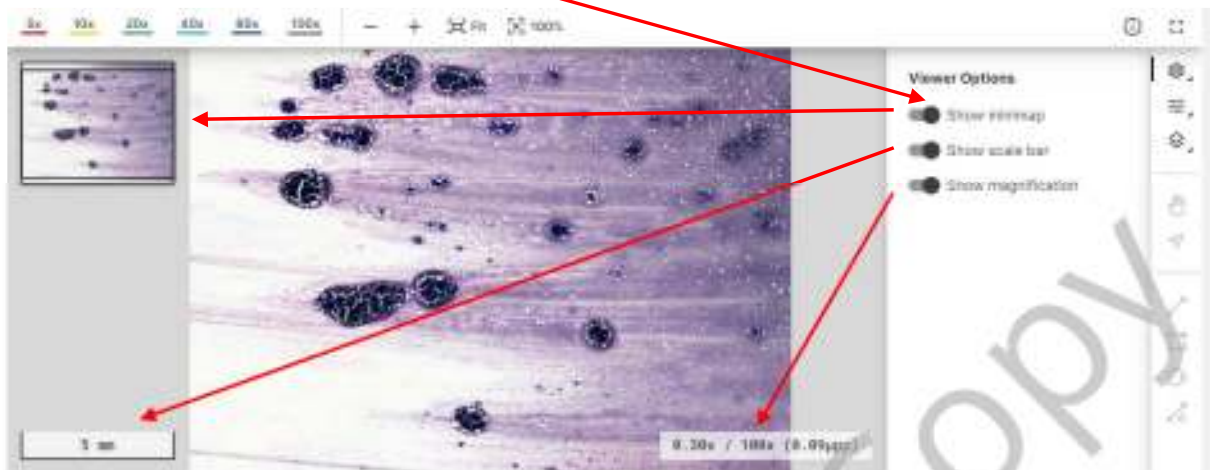

6. Clicking on the View Image filters icon allows for colour adjustment, brightness and sharpness

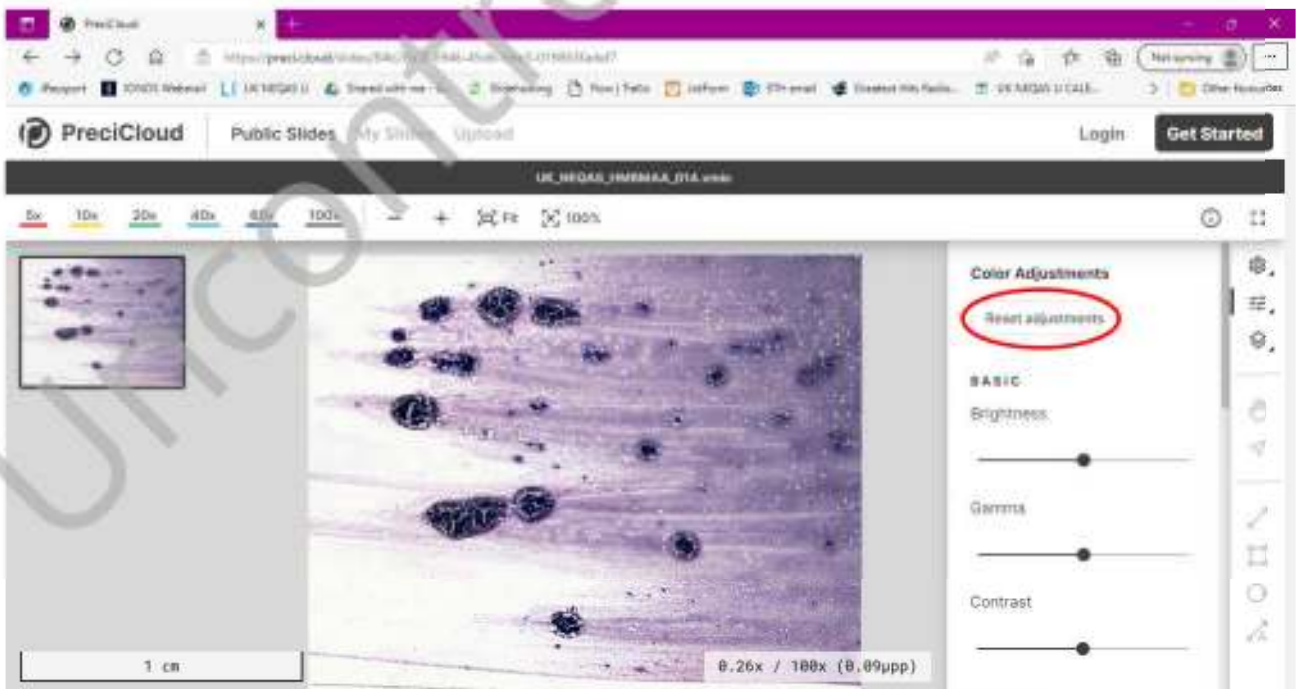

Please note that any changes will only affect the image on your screen and will not be saved to the image seen by other participants. Also, any adjustments can be reset


Identification of Cells A to E

7. Select the View and Import Annotations icon.



This will allow you to locate the 5 labelled cells for identification.

Please note that scrolling on the bar on the right (if present) will show the other cells for identification

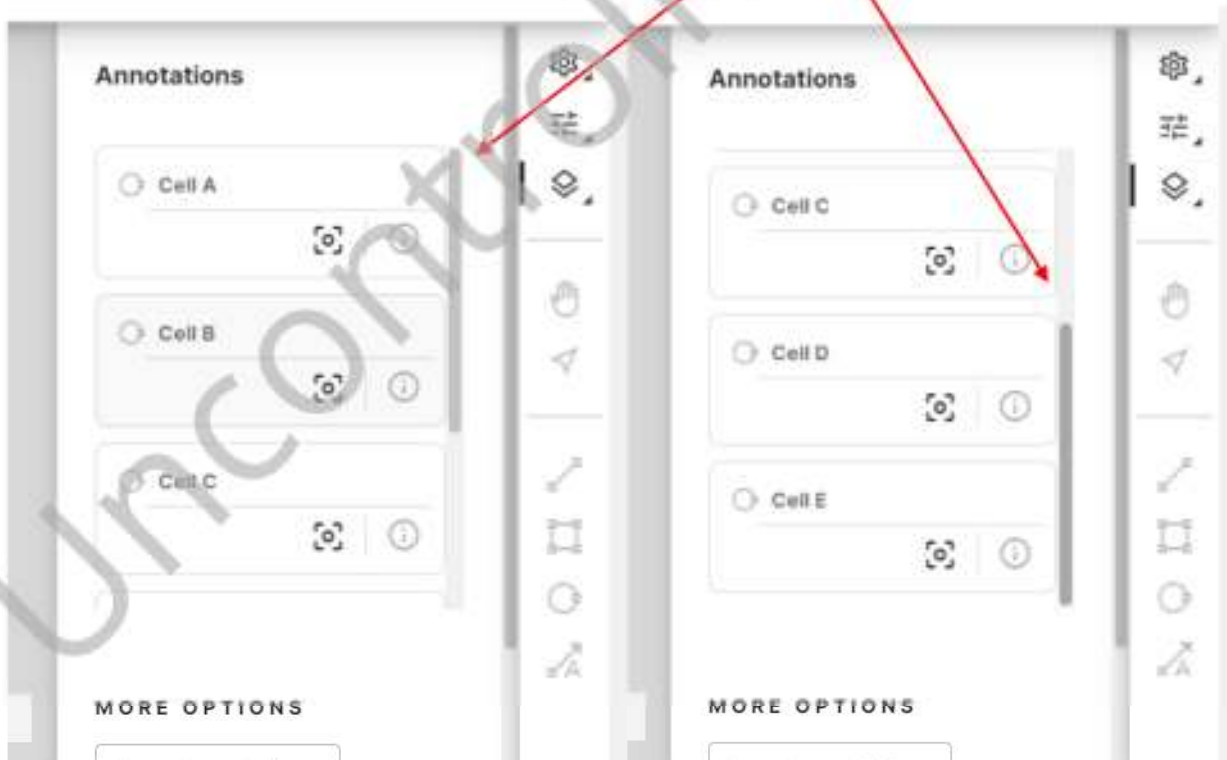

8. For each cell, click on the locate icon and the cell will automatically be selected (Please note the images shown are examples of cells A & B from the HMBMAA 212204 exercise)

